

Instructions

Charging

The wrap has a built-in battery that may be charged with the included USB > Type-C cable.

Charge the device for 2 hours. The device has a battery life of up to 1 hour of continuous use (or 4 applications).

If the lights begin to dim, it's time to charge.

Specifications

Power Consumption	56W
Charging Port	Type-C
Wavelength	660nm, 850nm
LED Quantity	112 * 0.5W
Net Weight	0.8 lb
Irradiance	120mW/ sq cm

Package Includes:

- 1 Red/NIR Light Hock Wrap
- 1 USB-Type-C cable
- 1 User Manual



Hock Helper User Manual

Contact/Customer Support
northeastequiwellness.com

Message (201) 279-1875

Leave a review on the website
or Facebook! It helps a ton!



Application

- The hock wrap can be used on either hock.
- Ensure the hole is facing backwards.
- Wrap snugly so that the LEDs are pressed against the skin for maximum efficacy.



Usage

- Press the power button once to activate both wavelengths.
- Press the power button twice to activate red light (660nm) only.
- Press the power button three times to activate near infrared light (850nm) only. This setting will not produce visible light as NIR light is invisible to the human eye.
- Press the power button one more time until the green light turns off. The device has powered off.
- Press the power button and hold for 3 seconds to activate the pulsing mode, then you can cycle through the settings again.

Usage Guidelines

- Never use over a cancerous growth.
- Do not use on recent burns.
- Do not use over the thyroid if the horse has hyperthyroidism.
- Consult a veterinarian before use on lame, ill, pregnant, or young horses.

